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User Guide

GTT@home

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Read this first.

Gwyliwch ein canllaw fideo yn:
gtt-at-home.com/video

Please watch our video guide at:
gtt-at-home.com/video

I gael cymorth, ffoniwch ein llinell gymorth:
0330 113 0275

For assistance, please call our help line:
0330 113 0275

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View our support information at:
gtt-at-home.com/care

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You can securely send your GTT@home test data to your midwife using your smartphone.

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Note: If you don't have a smartphone, you can send your GTT@home test data to your midwife using the provided pre-paid envelope (see page 20).

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There are three ways to download the GTT@home app:

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Scan the QR code below with your smartphone camera (recommended)

2. Agorwch borwr eich ffôn clyfar a theipiwch **gtt-at-home.com/app**

Open your smartphone's browser and type **gtt-at-home.com/app**

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Type '**GTT@home**' and hit search in your mobile app store

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This test will help your midwife assess how your body uses sugar.

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It does this by measuring your blood sugar levels before and after you take a sugar drink.

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The test kit contains everything you need to do the test and send a record of your test data back to your midwife for review.

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It takes **two hours and fifteen minutes** to complete the test.

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Note: Remember to fast overnight and do the test on waking.

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Eat nothing and drink only water until you have completed your test.

Beth sydd ei angen arnoch i wneud y prawf:

What you need to do the test:

Y Profwr The Tester

- 1** Wash your hands with warm water. Dry them.
- 2** Pull off protective cover A to switch on.
Remove cap from Pricker. Dispose of cap.
Press Pricker firmly against finger. Dispose of Pricker. Squeeze finger to form a blood drop this size
Place blood drop as shown until you hear a beep. The results will be sent to the blue data record.
- 3** Drink all of sugar drink provided.
- 4** After you take drink, press SET button. This sets a 2 hour timer.
Rest. Do not eat, drink or smoke.
When timer sounds press OFF button.
- 5** As before, wash your hands with warm water. Dry them.
- 6** Only open after 2 hours.
Pull off protective cover B.
Remove cap from an unused Pricker. Dispose of cap.
Press Pricker firmly against a finger. Dispose of Pricker. Squeeze finger to form a blood drop this size
Place blood drop as shown until you hear a beep. The results will be sent to the blue data record.
- 7** Test is complete when this light flashes.
Now detach this blue Data Record.
Follow the test kit instructions to return the test data.

Pigwyr Bys Finger Prickers

Sylwer: tab amddiffynol
Note: protective tab

Diod Glwcos Glucose Drink

Nodiadau pwysig

Important notes

Cofiwch ymprydio dros nos cyn gwneud y prawf.

Remember to fast overnight before doing the test.

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Eat nothing and drink only water until you have completed your test.

Mae'n cymryd tua 2 awr a 15 munud i wneud y prawf.

It takes approximately 2 hours and 15 minutes to do the test.

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Do not use a Finger Pricker if the protective tab has already been removed.

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Note: Do not open the Tester pouch until you do the test.

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Find somewhere comfortable to sit with a flat surface.

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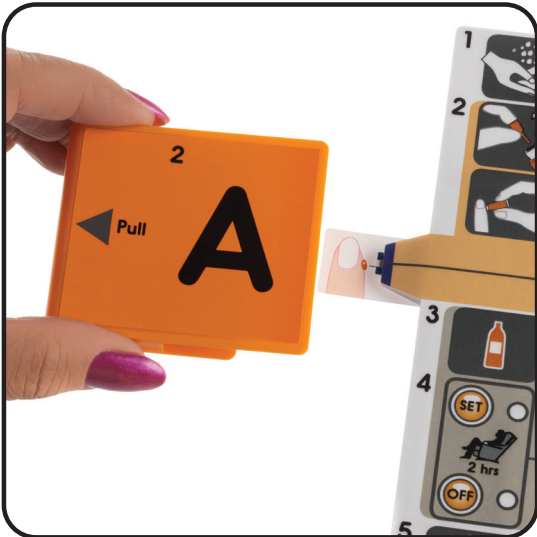
Remove protective cover A to switch on the Tester

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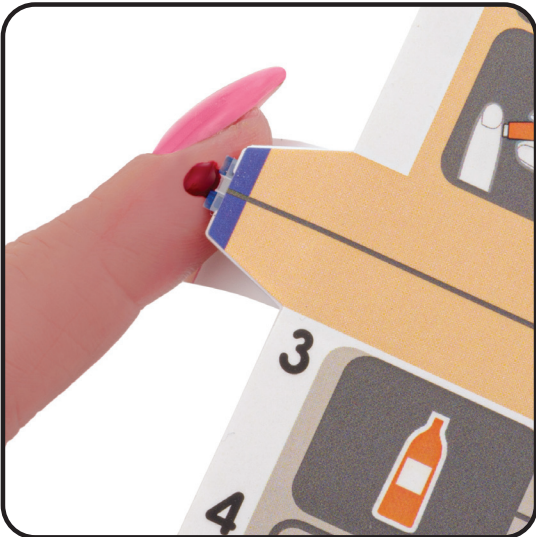
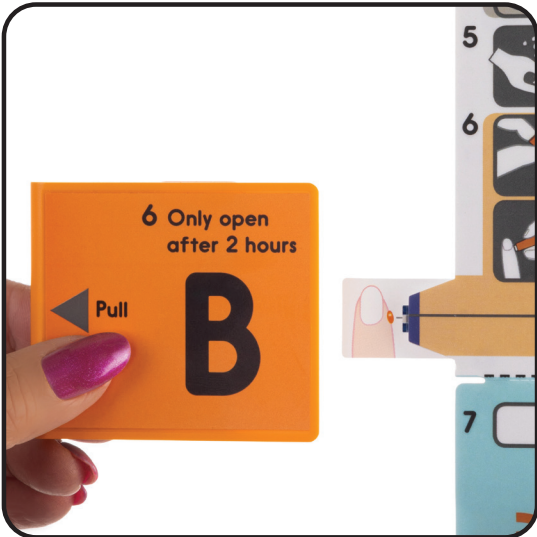
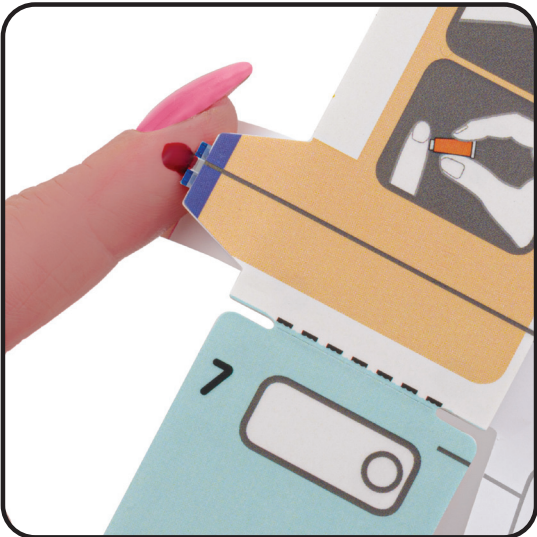
Note: Wait until the tester beeps and the light on the blue Data Record flashes.

- Mae'r data prawf bellach wedi cael eu cofnodi yn y Cofnod Data glas.
- The test data has now been recorded in the blue Data Record.
- Nawr gallwch dorri'r Cofnod Data glas i ffwrdd.
- Now you can snap off the blue Data Record.
- Plygwch y Cofnod Data i fyny ac i lawr nes y bydd yn torri i ffwrdd.
- Flex the Data Record up and down until it snaps off.

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NOTE: Make sure the blue Data Record is touching the back of your phone.

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Returning the blue Data Record by post

- Os na allwch sganio'r Cofnod Data glas gyda'ch ffôn clyfar, gallwch dychwelyd eich data prawf drwy'r post yn lle hynny.
- If you are unable to scan the blue Data Record with your smartphone, you can return your test data by post instead.
- Rhowch y Cofnod data glas yn yr amlen ragdaledig a'i phostio gan ddefnyddio blwch post Y Post Brenhinol.
- Place the blue Data Record in the pre-paid envelope and post it using a Royal Mail post box.
- Nid oes angen stamp – **mae postio'n rhad ac am ddim.**
- No stamp is needed – **postage is free.**

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This product is intended for the timed measurement of glucose in capillary blood. **Use immediately after opening The Tester pouch.**

Digostics Ltd

Harwell Innovation Centre, 173 Curie Avenue, Didcot, Oxfordshire, UK, OX11 0QG.

Product Name: GTT@home

REF DIG001

Digostics Ltd, Harwell Innovation Centre, Harwell, Didcot, Oxfordshire, OX11 0QG, United Kingdom

EC **REP**

Advena Limited, Tower Business Centre, 2nd Flr, Tower Street, Swatar, BKR 4013, Malta