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### **User Guide**

**GTT@home**

**GTT@home**

Pecyn Profi Goddefiad i Glwcos

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Glucose Tolerance Test Kit

**Read this first.**

**Gwylwch ein canllaw fideo yn:**

**[gtt-at-home.com/video](http://gtt-at-home.com/video)**

**Please watch our video guide at:**

**[gtt-at-home.com/video](http://gtt-at-home.com/video)**

I gael cymorth, ffoniwch ein llinell gymorth:

**0330 113 0275**

For assistance, please call our help line:

**0330 113 0275**

Gweler ein gwybodaeth gymorth yn

**[gtt-at-home.com/care](http://gtt-at-home.com/care)**

View our support information at:

**[gtt-at-home.com/care](http://gtt-at-home.com/care)**

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You can securely send your GTT@home test data to your midwife using your smartphone.

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**Note: If you don't have a smartphone, you can send your GTT@home test data to your midwife using the provided pre-paid envelope (see page 20).**

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### There are three ways to download the GTT@home app:

1. Sganiwch y cod QR isod gyda chamera eich ffôn clyfar (argymhellir)
1. Scan the QR code below with your smartphone camera (recommended)

2. Agorwch borwr eich ffôn clyfar a theipiwch **gtt-at-home.com/app**
2. Open your smartphone's browser and type **gtt-at-home.com/app**
3. Teipiwch '**GTT@home**' a chlicio chwilio yn eich siop apiau symudol
3. Type '**GTT@home**' and hit search in your mobile app store

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**This test will help your midwife assess how your body uses sugar.**

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It does this by measuring your blood sugar levels before and after you take a sugar drink.

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The test kit contains everything you need to do the test and send a record of your test data back to your midwife for review.

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It takes **two hours and fifteen minutes** to complete the test.

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**Note: Remember to fast overnight and do the test on waking.**

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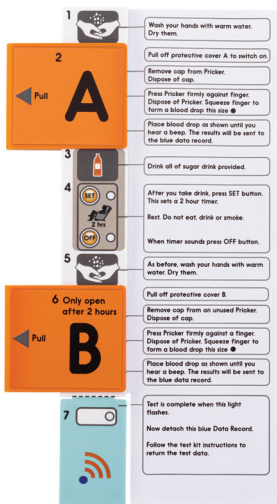
**Eat nothing and drink only water until you have completed your test.**

# Beth sydd ei angen arnoch i wneud y prawf:

What you need to do the test:

## Y Profwr The Tester

## Pigwyr Bys Finger Prickers

Sylwer: tab amddiffynnol

Note: protective tab

## Diod Glwcos Glucose Drink

# Nodiadau pwysig

## Important notes

**Cofiwch ymprydio dros nos cyn gwneud y prawf.**

**Remember to fast overnight before doing the test.**

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**Eat nothing and drink only water until you have completed  
your test.**

**Mae'n cymryd tua 2 awr a 15 munud i wneud y  
prawf.**

**It takes approximately 2 hours and 15 minutes  
to do the test.**

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**Do not use a Finger Pricker if the protective tab has  
already been removed.**

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**Note: Do not open the Tester pouch until you do the test.**

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1. Find somewhere comfortable to sit with a flat surface.

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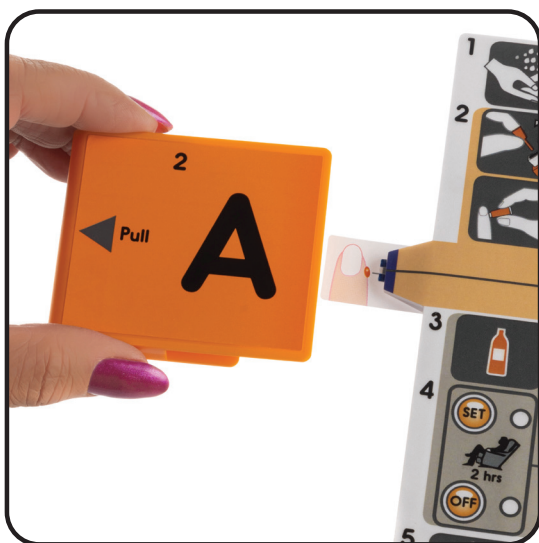
Remove protective cover A to switch on the Tester

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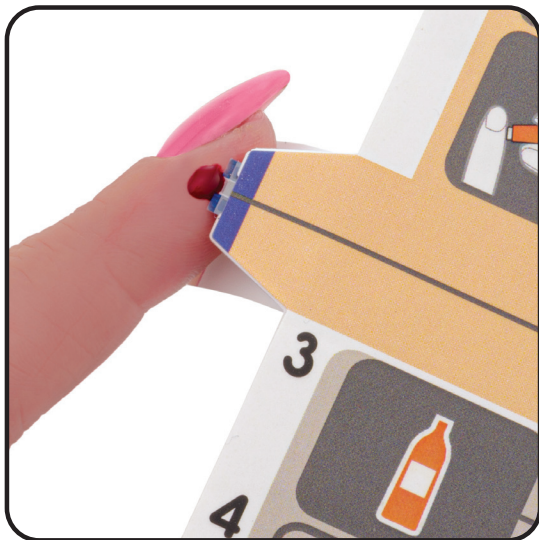
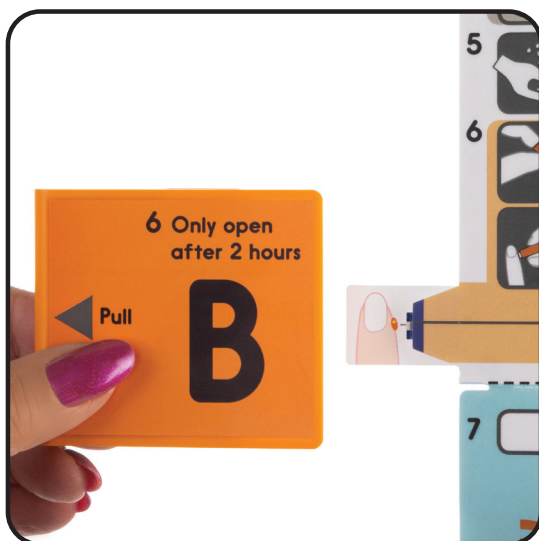
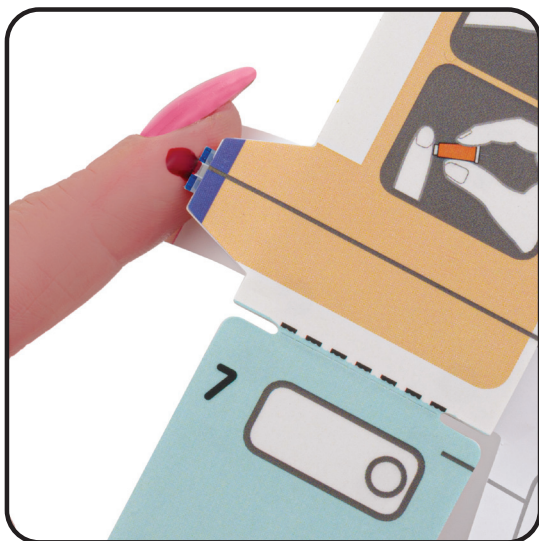
**Note: Wait until the tester beeps and the light on the blue Data Record flashes.**

- Mae'r data prawf bellach wedi cael eu cofnodi yn y Cofnod Data glas.
- The test data has now been recorded in the blue Data Record.
- Nawr gallwch dorri'r Cofnod Data glas i ffwrdd.
- Now you can snap off the blue Data Record.
- Plygwch y Cofnod Data i fyny ac i lawr nes y bydd yn torri i ffwrdd.
- Flex the Data Record up and down until it snaps off.

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**NOTE: Make sure the blue Data Record is touching the back of your phone.**

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## Returning the blue Data Record by post

- Os na allwch sganio'r Cofnod Data glas gyda'ch ffôn clyfar, gallwch ddychwelyd eich data prawf drwy'r post yn lle hynny.
- If you are unable to scan the blue Data Record with your smartphone, you can return your test data by post instead.
- Rhowch y Cofnod data glas yn yr amlen ragdaledig a'i phostio gan ddefnyddio blwch post Y Post Brenhinol.
- Place the blue Data Record in the pre-paid envelope and post it using a Royal Mail post box.
- Nid oes angen stamp – **mae postio'n rhad ac am ddim.**
- No stamp is needed – **postage is free.**

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This product is intended for the timed measurement of glucose in capillary blood. **Use immediately after opening The Tester pouch.**

**Digostics Ltd**

Harwell Innovation Centre, 173 Curie Avenue, Didcot, Oxfordshire, UK, OX11 0QG.

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**REF** DIG001

**Digostics Ltd**, Harwell Innovation Centre, Harwell, Didcot, Oxfordshire, OX11 0QG, United Kingdom

**EC** **REP**

**Psephos Limited**, ATU iHub Galway, Dublin Road, Galway, H91 DCH9, Ireland