User Guide

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0330 113 0275

For assistance, please call our help line:

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You can securely send your GTT@home test data to your midwife using your
smartphone.

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Note: If you don't have a smartphone, you can send your GTT@home test
data to your midwife using the provided pre-paid envelope (see page 20).

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There are three ways to download the GTT@home app:

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1. Scan the QR code below with your smartphone camera (recommended)

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2. Open your smartphone's browser and type **gtt-at-home.com/app**

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3. Type 'GTT@home' and hit search in your mobile app store

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For assistance, contact
gtt-at-home.com/support.

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This test will help your midwife assess how your body uses sugar.

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It does this by measuring your blood sugar levels before and after you take a sugar drink.

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The test kit contains everything you need to do the test and send a record of your test data back to your midwife for review.

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It takes **two hours and fifteen minutes** to complete the test.

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Note: Remember to fast overnight and do the test on waking.

Eat nothing and drink only water until you have completed your test.

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What you need to do the test:

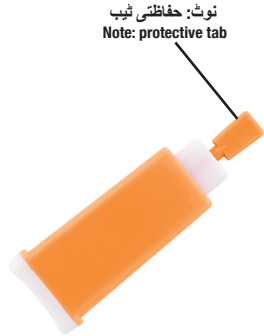
- 1 Wash your hands with warm water. Dry them.
- 2 Pull off protective cover A to switch on.
Remove cap from Pricker. Dispose of cap.
Press Pricker firmly against finger. Dispose of Pricker. Squeeze finger to form a blood drop this size ●
Place blood drop as shown until you hear a beep. The results will be sent to the blue data record.
- 3 Drink all of sugar drink provided.
- 4 After you take drink, press SET button. This sets a 2 hour timer.
Rest Do not eat, drink or smoke.
When timer sounds press OFF button.
- 5 As before, wash your hands with warm water. Dry them.
- 6 Only open after 2 hours
Pull off protective cover B.
Remove cap from unused Pricker. Dispose of cap.
Press Pricker firmly against a finger. Dispose of Pricker. Squeeze finger to form a blood drop this size ●
Place blood drop as shown until you hear a beep. The results will be sent to the blue data record.
- 7 Test is complete when this light flashes.
Now detach this blue Data Record.
Follow the test kit instructions to return the test data.

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Remember to fast overnight before doing the test.

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It takes approximately 2 hours and 15 minutes to do the test.

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Do not use a Finger Pricker if the protective tab has already been removed.

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Note: Do not open the Tester pouch until you do the test.

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1. Find somewhere comfortable to sit with a flat surface.

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• Your hands must be clean to produce good test results.

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• Using warm water will make it easier for you to make a blood drop.

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Remove protective cover A to switch on the Tester

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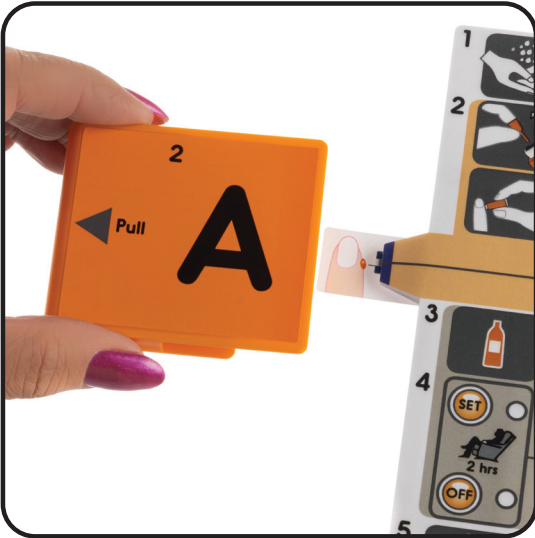
• Slide off cover A by pulling it as shown.

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• The Tester will beep as it switches on.

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• After you have pricked your finger, make a blood drop about this size.

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• Gently massage the finger to increase the size of the blood drop.

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• Put your blood drop on to the TIP of the Tester as shown.

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• The Tester beeps when correctly filled.

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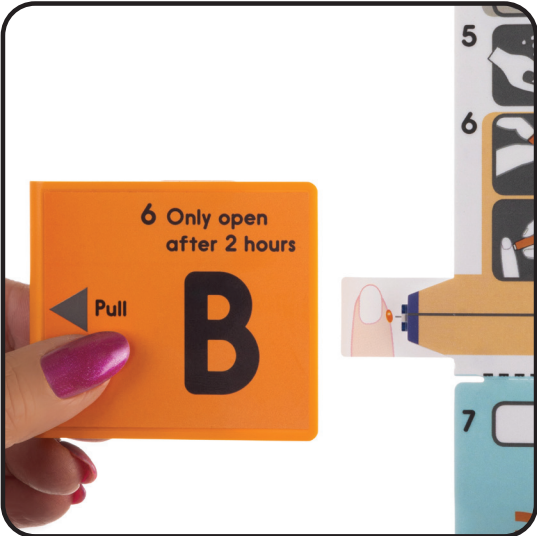
• Slide off cover B by pulling it as shown.

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• Twist and then pull off the protective tab.

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• Press the Pricker firmly against your finger as shown in the photograph.

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• The Pricker will automatically prick your finger.

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- After you have pricked your finger, make a blood drop about this size.

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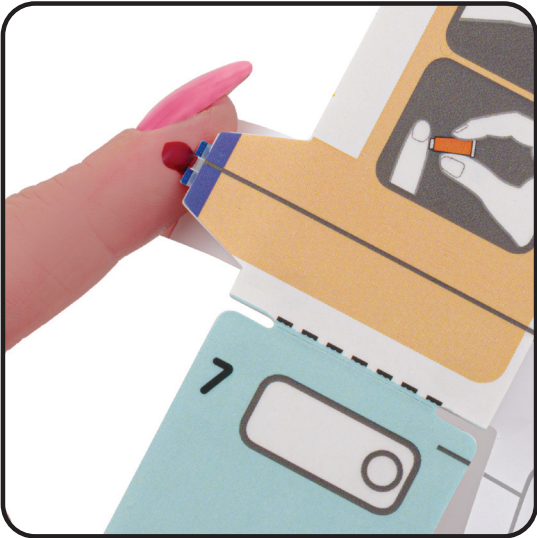
- Gently massage the finger to increase the size of the blood drop.

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- Put your blood drop on to the TIP of the Tester as shown.

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- The Tester beeps when correctly filled.

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Note: Wait until the tester beeps and the light on the blue Data Record flashes.

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• The test data has now been recorded in the blue Data Record.

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• Now you can snap off the blue Data Record.

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• Flex the Data Record up and down until it snaps off.

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NOTE: Make sure the blue Data Record is touching the back of your phone.

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Returning the blue Data Record by post

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• If you are unable to scan the blue Data Record with your smartphone, you can return your test data by post instead.

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• Place the blue Data Record in the pre-paid envelope and post it using a Royal Mail post box.

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This product is intended for the timed measurement of glucose in capillary blood.

Use immediately after opening The Tester pouch.

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Digostics Ltd

Harwell Innovation Centre, 173 Curie Avenue, Didcot,
Oxfordshire, UK, OX11 0QG.

Product Name: GTT@home

REF

DIG001

Digostics Ltd, Harwell Innovation Centre, Harwell,
Didcot, Oxfordshire, OX11 0QG, United Kingdom

EC

REP

Advena Limited, Tower Business Centre,
2nd Flr, Tower Street, Swatar, BKR 4013, Malta